



MALDI Biotyper®

MBT Biotarget 96 US IVD

Cost-effective and convenient

Innovation with Integrity

Features and benefits

- Disposable MALDI target plates
- 96 sample positions
- Unique barcode for traceability
- Perfectly flat surface for reliable and reproducible results
- High flexibility in sample run sizes
- No wasted spots
- AnchorChip™ effect facilitates preparation

US-IVD

MBT Biotarget 96

A disposable target for microorganism identification in the clinical MALDI Biotyper CA System workflow.

Easy to use and convenient

The MBT Biotargets are equipped with the Bruker AnchorChip™ technology. A hydrophobic coating on the target surrounding the hydrophilic sample positions facilitates the preparation of samples, as liquids concentrate into the spots, hence avoiding bleeding to neighboring spots. Disposable MBT Biotargets are always ready-to-use because there is no need to clean.

Flexible and cost-effective – No wasted spots

The 96 sample position format provides high flexibility in sample run sizes. The perfectly flat surface eliminates the need for predefined calibrant positions and offers flexibility to apply the calibrant anywhere on the target. This allows to use all spots, no matter how long the individual sample run is. Via the unique target barcode, the system knows which spots can still be utilized, resulting in maximum target usage with minimal waste. Additionally, the blue color helps to visualize clearly the dried spot positions already used during a previous run.

Disposable targets increase your lab's efficiency by eliminating the labor time involved in cleaning reusable targets and maintaining cleaning supplies.

Order information

MBT Biotarget 96 US IVD

Part No. 1840380

Set of 20 individually barcoded targets; total 1,920 sample positions

Part No. 8267615


Adapter (one piece) required to use MBT Biotargets with MALDI Biotyper systems

Part No. 8270006

Set of 10 MBT Biotarget transport boxes

Care-free handling and transportation

Laboratories that transport prepared MALDI target plates benefit from simplified stock management by negating the need to maintain costly return schemes for reusable MALDI target plates. The robustness of the MBT Biotargets ensures safe shipment.

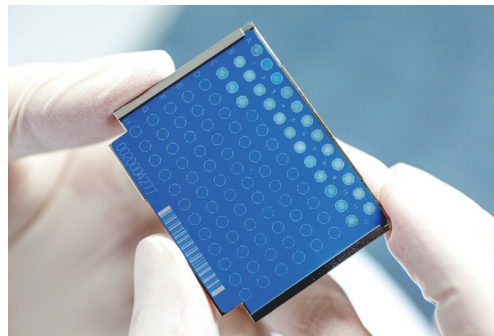

The MBT Biotarget 96 US IVD requires the use of MALDI Biotyper CA System Software (version Claim 3 or higher).

Only for sale in the USA and Puerto Rico.

IVD

Rx ONLY

MALDI Biotyper® is a registered trademark of the Bruker group of companies.

Online information

bruker.com/microbiology



Bruker Daltonics GmbH and Co. KG

Bremen · Germany
Phone +49 (0) 421-2205-0

info.md@bruker.com

Bruker Scientific LLC

Billerica, MA · USA
Phone +1 (978) 663-3660

